









Biocard® HBsAg RAPID TEST KIT


Biocard®HBsAg Rapid Test Kit is an Immunochromatographic assay for the visual detection of Hepatitis B surface antigen (HBsAg)

-  Approved by National Institute Biologicals (NIB)
-  Detection Limit: 0.5 ng/ml
-  Presence of control line to validate the assay
-  Sample types: Serum, Plasma (70µL)
-  Pre-coated monoclonal anti-HBsAg antibody
-  Sensitivity : 99.3% Specificity : 99.2%
-  Results with in 20 minutes
-  Storage: 4°C to 30°C




Biocard® HBsAg RAPID TEST KIT

Test Procedure

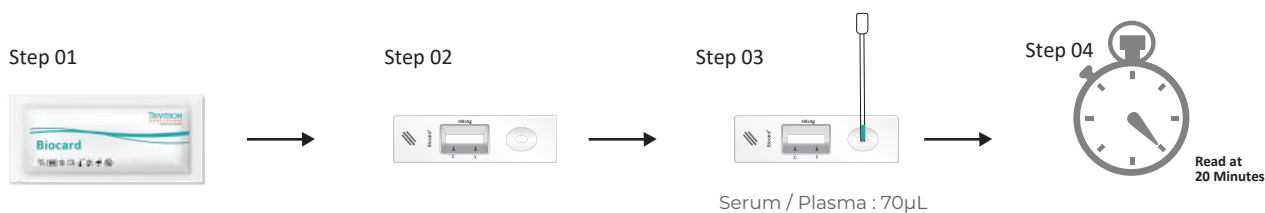

Result Interpretation

For Biocard® HBsAg Device

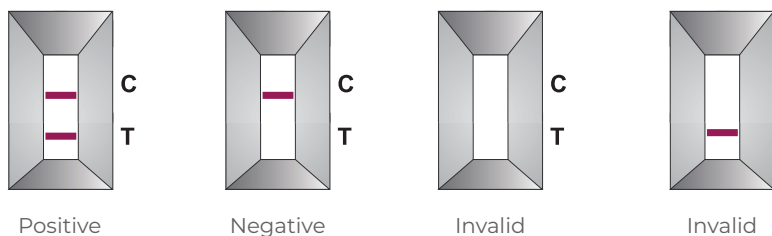

Performance

No of Positive Samples	True Positives	False Negatives
318	316	2
No of Negative Samples	True Negatives	False Positives
1797	1784	13

Diagnostic Sensitivity: 93.3 %

Diagnostic Specificity: 99.2 %

Product Code	Product Name	Pack Size	Storage	Shelf Life
30201138	Biocard® HBsAg RAPID TEST KIT	30 Test	4-30°C	18 Months

*Please note: Product specifications are subject to change without prior notice owing to product modifications, improvements / up-gradation.



corporate@triviron.com | www.triviron.com

Customer Care Helpline: +91 98400 80008

